


3M™ Scotchcast™ Splinting Guide




Ulnar Gutter Splint

Common Uses
5th Metacarpal Fracture, 4th Metacarpal Fracture

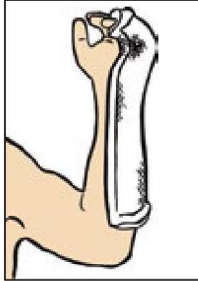


Volar Splint

Common Uses
Wrist Sprains/Strains, Carpal Tunnel, Lacerations, Night Splints

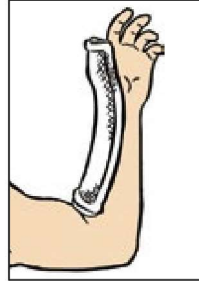


Thumb Spica Splint

Common Uses
Navicular Fracture, Thumb Dislocation, Ligamentous Injuries, Bennett's Fracture

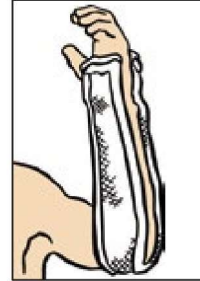


Sugar Tong Splint

Common Uses
Colles' Fracture, Forearm Fracture (Radial/Ulnar)

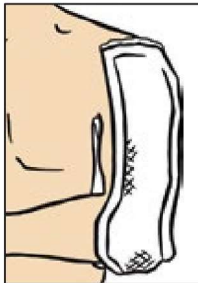


Sugar Tong Splint

Common Uses
Humeral Fracture

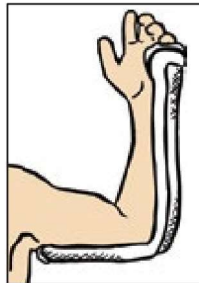


Posterior Long-Arm Splint

Common Uses
Supracondylar Fracture, Elbow Sprains/Strains

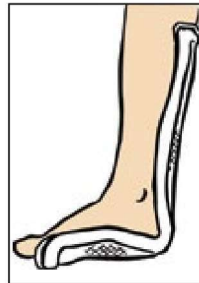


Posterior Short-Leg Splint

Common Uses
Tibia/Fibula Fracture, Ankle Fracture, Metatarsal Fracture

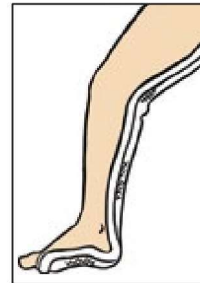


Posterior Long-Leg Splint

Common Uses
Ligamentous Injuries of the Knee, Dislocation of the Knee Joint, Tibia and Fibula Shaft Fractures, Femoral Shaft Fracture

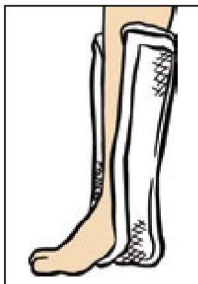

Stirrup Splint

Common Uses
Ankle Fractures, Ankle Sprains/Strains

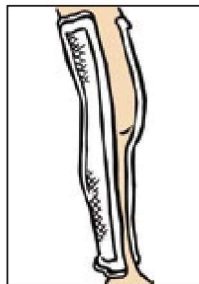


Medial-Lateral Long-Leg Splint

Common Uses
Ligamentous Injuries of the knee, Tibia and Fibula Fractures

Splinting Wetting Techniques




Water Bottle Technique

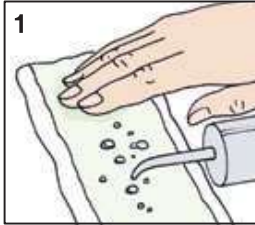


1 Dribble water onto backing material and rub beaded water to penetrate backing material.

Water Bottle/Faucet Technique

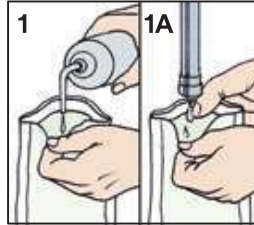


1 Pour minimal amount of water down back side of fiberglass using faucet or water bottle to dampen splint.

Opening Splint Cover Technique

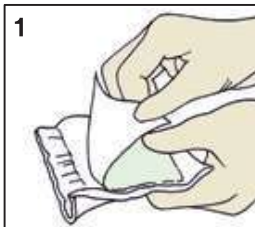


1 Peel back one side of backing material to expose splint.

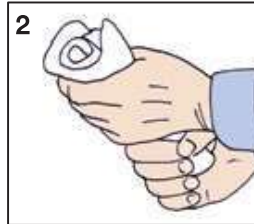

2 Squeeze splint to spread water into fiberglass.


Dipping in Bucket Technique

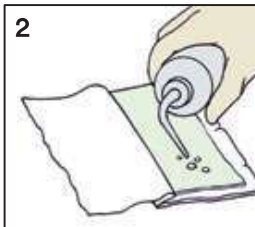


2 Dribble water onto exposed splinting material.



1 Squeeze one or two times while immersed in water.

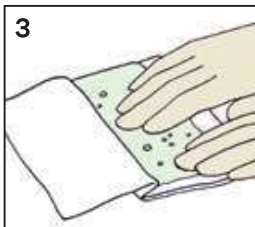


3 Rub water to penetrate splint material.

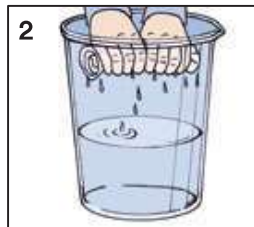

2 Squeeze out excess water.



4 Replace backing material.

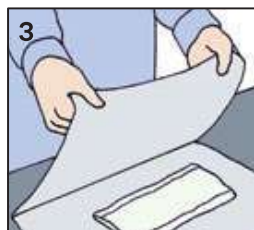

3 Wrap flat in towel and press to blot out remaining excess water.

Important note: Refer to the Instructions for Use for complete information related to the use of 3M™ Scotchcast™ splinting products.



3M Health Care
Infection Prevention Division
2510 Conway Avenue
St. Paul, MN 55144-1000
USA
Phone 1-800-228-3957
Web 3M.com/immobilization

[View training videos](#)

[Sign up for a workshop](#)

3M and Scotchcast are trademarks of 3M. Used under license in Canada. Please recycle. Printed in USA. © 3M 2016. All rights reserved. 70-2009-0723-9